




MAGNETIC BUCKLE
BOUCLE MAGNÉTIQUE

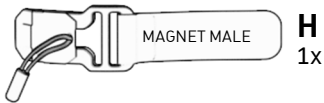

XTERN FRONTIER SOFT PAD & STRAP KIT SMALL-MEDIUM-LARGE KIT COUSSIN COURROIE PETIT-MOYEN-GRAND




(502400-K00-SML)

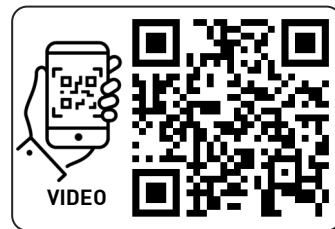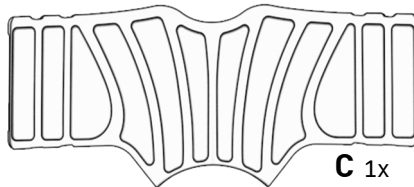


FREE LIBRE **D 1x** FIXED FIXE


-1-

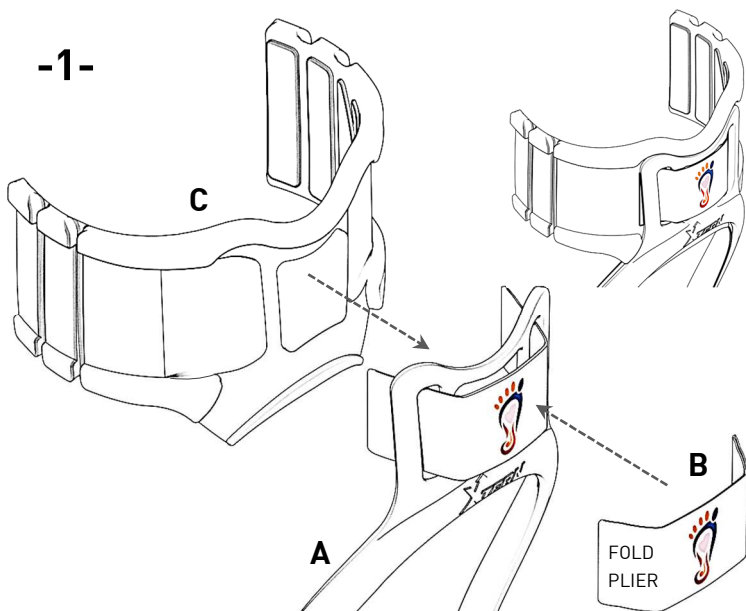


-2-

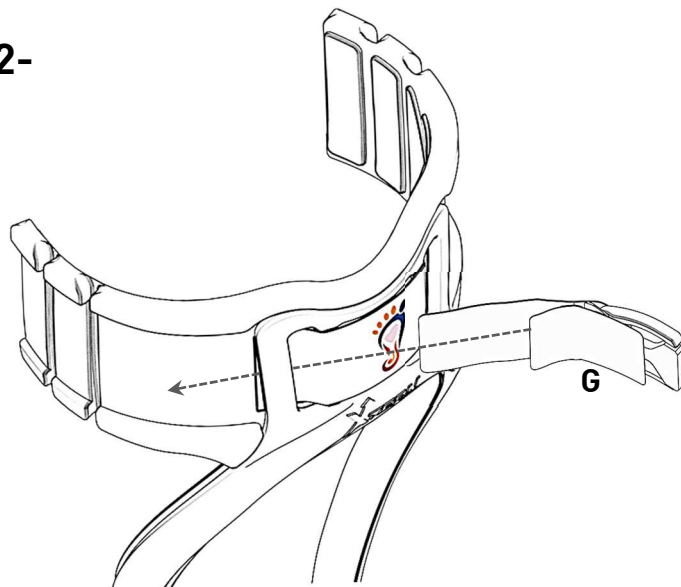


-3-

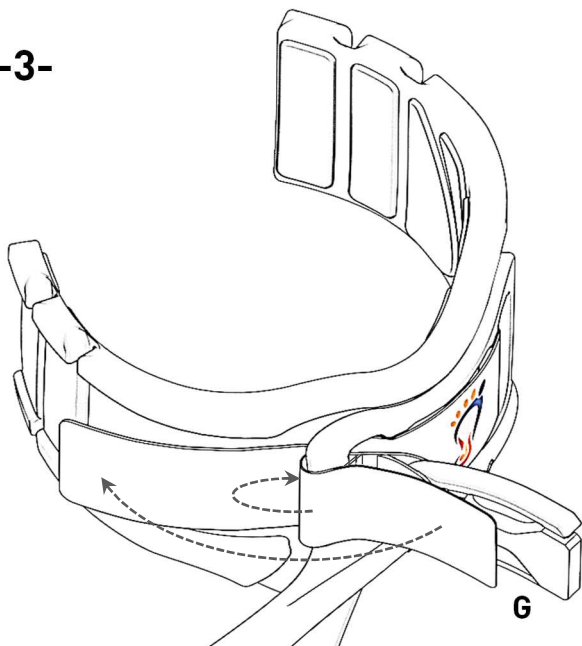


-4-

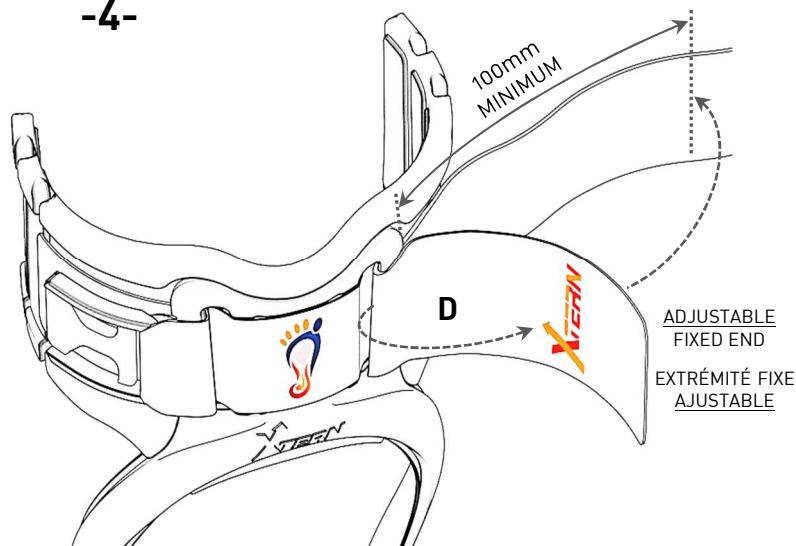



-5-

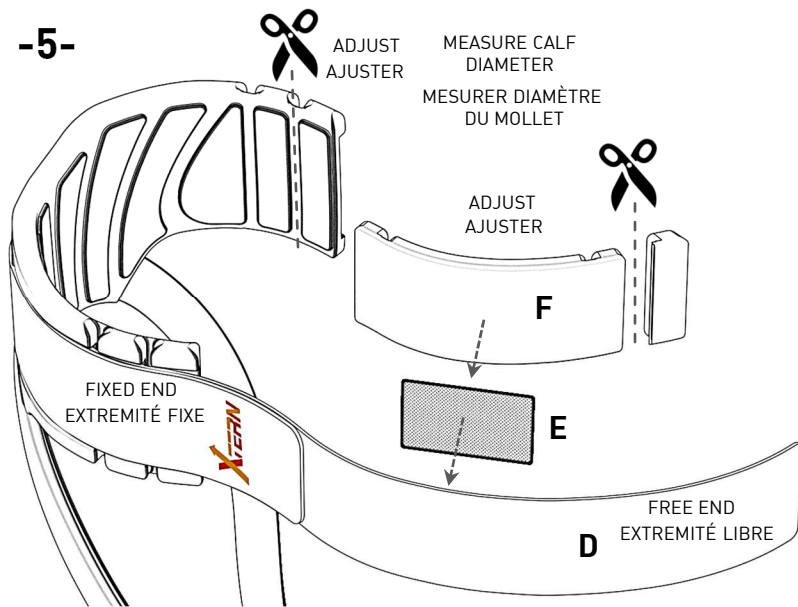


-6-

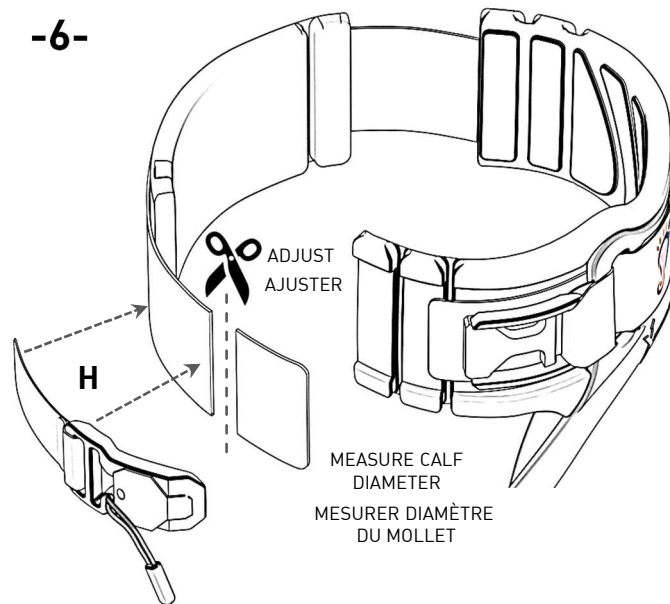

-7-

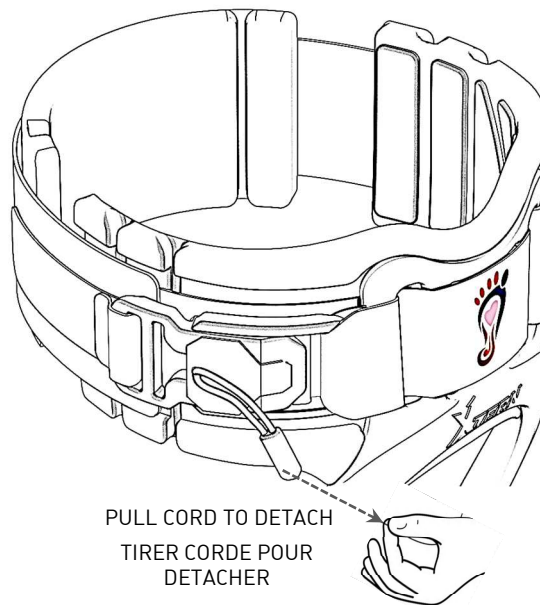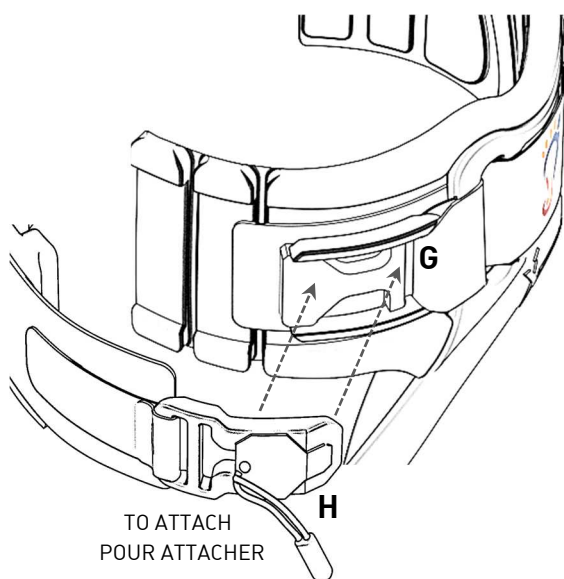

-8-

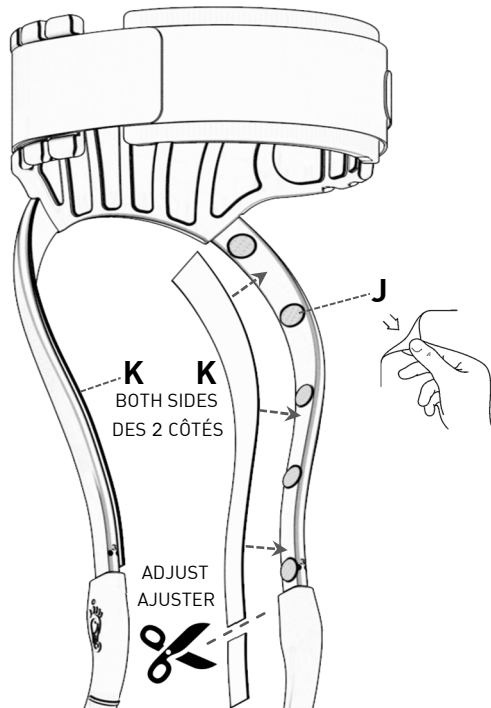

⚠

**IMPORTANT SAFETY VERIFICATION
TO DO PRIOR EVERY USE !**

FINAL USER MUST ALWAYS VERIFY AND INSURE
THAT MAGNET BUCKLE MECHANISM
IS FULLY ENGAGED AND LOCKED IN SAFELY
MANNER BEFORE USING THE DEVICE.

FAILING TO PROCEED TO THIS VERIFICATION COULD
LEAD TO UNPLANNED OPENING OF MAGNETIC
BUCKLE THAT COULD CAUSE THE USER TO FALL
AND CAUSE SERIOUS INJURIES !

⚠

**VÉRIFICATION DE SECURITÉ IMPORTANTE
À FAIRE AVANT CHAQUE USAGE!**

L'UTILISATEUR FINAL DOIT TOUJOURS VÉRIFIER ET
S'ASSURER QUE LE MÉCANISME DE BOUCLE
MAGNÉTIQUE EST COMPLÈTEMENT ENCLÉNCÉ ET
FERMÉ DE FAÇON SÉCURITAIRE AVANT
D'UTILISER L'APPAREIL.

NE PAS PROCÉDER À CETTE VÉRIFICATION
POURRAIS PROVOQUER UN OUVERTURE IMPRÉVUE
DE LA BOUCLE MAGNÉTIQUE QUI POURRAIS
ENTRAÎNER UNE CHUTE DE L'UTILISATEUR ET
CAUSER DES BLESSURES SÉRIEUSES !

